


Essederm Lips+

Essederm® Lips+ is a "next-generation" of dermal filler that is made from hyaluronic acid – a naturally occurring substance that plays a vital role in hydrating your skin and replenishing its natural volume. No allergy skin tests are needed prior to receiving treatment

Essederm® Lips+ is a sterile restorable cross-linked pyrogen free hyaluronic acid gel of non-animal origin which is used for: Filling superficial wrinkles (perioral, periorbital...), Augmentation of lip and injection in lip contour.

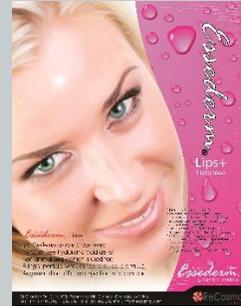

Specifications:

- 1.0 ml cross link hyaluronic acid
- 27 G disposable glass syringe
- It is sterile pyrogen free, colorless, isotonic gels, transparent, odorless,
- viscous, no particle and 100% none animal base
- Cross linked NaHA content 20 mg per ml
- Not cross linked NaHA content up to 1.2 mg per ml
- Cross linking grade: up to 90 %
- BDDE Content: < 0.02 mg per ml
- Lidocaine (.3 Mg/ml)ML
- Storage and condition: 2 to 25 C (35 to 77 F)

Treatment Area:

- Lip Augmentation

Injection Techniques:


For a cosmetic microplasty treatment, we advise to use the following injection techniques:

Serial puncture technique:

Make a series of injections along the length of the wrinkle. The spots of Essederm® Lips+ under the skin should join up to form a continuous line. If there is over-correction, massage firmly immediately after injection. This injection technique is particularly suitable for filling in micro depressions and small wrinkles with Essederm® Lips+.

Linear threading technique:

Introduce the whole needle into the Lips area at an angle of about 30°.

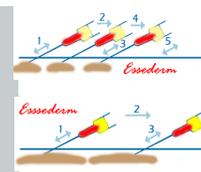

Essederm Deep+

Essederm® Deep+ is a "next-generation" of dermal filler that is made from hyaluronic acid – a naturally occurring substance that plays a vital role in hydrating your skin and replenishing its natural volume. No allergy skin tests are needed prior to receiving treatment.

Essederm® Deep+ is a sterile restorable cross-linked pyrogen free hyaluronic acid gel of non-animal origin which is used for: Deep+ cutaneous depressions like Deep+ wrinkles, folds and other defects, restoring or creating volume in facial contours (cheekbones, chin).

Specifications:

- 1.0 ml cross link hyaluronic acid
- 27G disposable glass syringe
- It is sterile pyrogen free, colorless, isotonic gels, transparent, odorless, viscous, no particle and 100% none animal base
- Cross linked NaHA content 17 to 19 mg per ml
- Not cross linked NaHA content up to 1.2 mg per ml
- Cross linking grade: up to 90 %
- BDDE Content: < 0.02 mg per ml
- Lidocaine (.3 mg/ml)
- Storage and condition: 2 to 25 C (35 to 77 F)

Treatment Area

- Augmentation of Cheekbones, Essederm Deep+
- Cheek Augmentation / Cheek Wrinkles
- Nasolabial Folds
- Oral Commissure / Smile Lines
- Chin Augmentation

Injection Techniques




For a cosmetic microplasty treatment we advise to use the following injection techniques:

Sandwich technique

For Deep+er cutaneous depressions, first inject into the Deep+ dermis at an angle of about 30° to the plane of the epidermis, then into the middle dermis at an angle of about 15°. This technique is particularly suitable for filling in clefts and Deep+ glabellae.

Fan technique

In order to enhance volume over a broader surface without increasing the number of dermal injection sites, use the fan technique. Introduce the needle at the edge of the zone to be treated in the same way as for the retro-tracing technique. After injecting a line of the product, instead of withdrawing the needle from the skin, reinsert it at a different angle and inject again as before. Repeat the operation to obtain a fan-shaped series of lines.

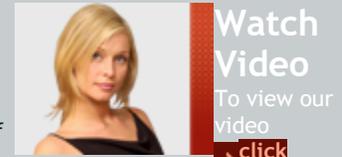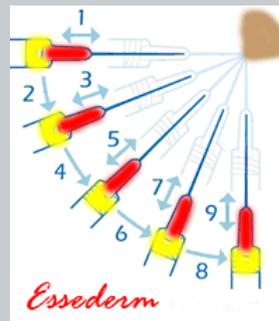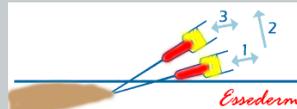

[Back](#)

Medium+

Medium+® Medium is a "next-generation" of dermal filler that is made from hyaluronic acid – a naturally occurring substance that plays a vital role in hydrating your skin and replenishing its natural volume. No allergy skin tests are needed prior to receiving treatment

Medium+® Medium is a Sterile restorable cross-linked pyrogen free hyaluronic acid gel of non-animal origin which is used for: Filling medium and deep cutaneous depressions such as wrinkles, folds and other defects, restoring or creating volume in facial contours (cheekbones, chin) and lip contours.

Specifications:

- 1.0 ml cross link hyaluronic acid
- 27G disposable glass syringe
- It is sterile pyrogen free, colorless, isotonic gels, transparent, odorless, viscous, no particle and 100% none animal base
- Cross linked NaHA content 20 mg per ml
- Not cross linked NaHA content up to 1.2 mg per ml
- Cross linking grade: up to 90 %
- BDDE Content: < 0.02 mg per ml
- Lidocaine (.3 mg/ml)
- Storage and condition: 2 to 25 C (35 to 77 F)

Treatment Area:

- Worry Lines
- Glabellar Lines
- Augmentation of Cheekbones
- Cheek Augmentation / Cheek Wrinkles
- Nasolabial Folds
- Vermilion Border
- Oral Commissure / Smile Lines
- Lip Contour
- Chin Augmentation

Injection Techniques



For a cosmetic microplasty treatment we advise to use the following injection techniques:

Serial puncture technique

Make a series of injections along the length of the wrinkle. The spots of Medium+® Medium under the skin should join up to form a continuous line. If there is over-correction, massage firmly immediately after injection. This injection technique is particularly suitable for filling in micro depressions and small wrinkles with Medium+® Medium.

Linear threading technique

Introduce the whole needle into the wrinkle at an angle of about 30°. The orifice of the needle should be pointing up for injections into the middle and deep dermis (Medium+, Deep+ and Subline+), but down for injection into the superficial dermis (Medium+). The Medium+® is injected while the needle is being gently pulled back.

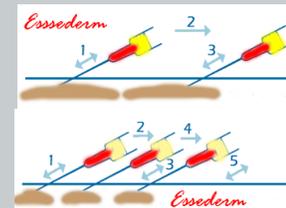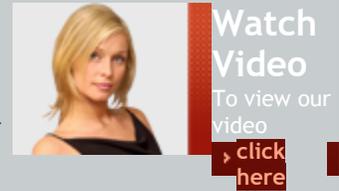

[Back](#)


Essederm Subline+

Essederm® Subline+ is a sterile restorable cross-linked pyrogen free with Lidocaine hyaluronic acid gel of non-animal origin which is used for: deep skin deformations and depressions, increasing volume and contours of the face (chin, cheek bones), rhinoplasty and scar correction.

Essederm® Subline+ is a high-viscosity implant which is therefore designed for implantation into the deep dermis.

Essederm® Subline+ is indicated for prevention or as an adjunct to cosmetic and reparative surgery. Essederm® Subline+ is the most highly concentrated hyaluronic acid-based filling Subline+duct on the market. Exceptional viscoelasticity for a treatment without canulas.

For the biggest-volume correction operations, use the cross-hatching technique. With the needle at an angle of 30° to the plane of the skin throughout, perform four parallel linear threading injections and then another four perpendicular to the first series. This type of injection technique is ideal for filling of the cheeks.

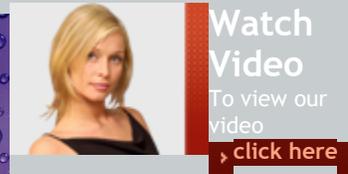

Specifications:

1.0 ml cross link hyaluronic acid

27G disposable glass syringe

It is sterile pyrogen free, colorless, isotonic gels, transparent, odorless, viscous, no particle and 100% none animal base

Cross linked NaHA content 20 mg per ml

Not cross linked NaHA content up to 1.2 mg per ml

Cross linking grade: up to 90 %

BDDE Content: < 0.02 mg per ml

Lidocaine (.3 mg/ml)

Storage and condition: 2 to 25 C (35 to 77 F)

Treatment Area

Augmentation of Cheekbones

Cheek Augmentation / Cheek Wrinkles,

Nasolabial Folds,

Chin Augmentation




Injection Techniques

For a facial volumetric treatment we advise to use the following injection techniques:

Cross-hatching technique

For the biggest-volume correction operations, use the cross-hatching technique. With the needle at an angle of 30° to the plane of the skin throughout, perform four parallel linear threading injections and then another four perpendicular to the first series. This type of injection technique is ideal for filling of the cheeks.

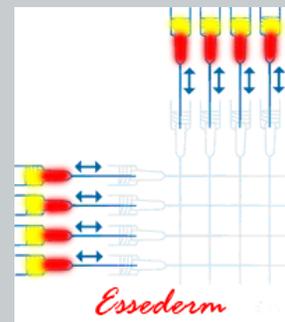

Back